

SKIN MARKER USAGE

1. Cleanse the skin thoroughly with soap and water. Pat the skin dry. Take the white (fine) micro-brush and dip it into the Softap pigment. Draw on the one eyebrow. After an agreement on the shape , you may draw on the other eyebrow .



2. Take the skin marker and hold it perpendicular to the skin. Make sure that the skin marker is at a 90 degree angle to the skin. This will ensure fine points, versus large uncontrolled points. On the outside edge of the brow design, make a small dotted line directly adjacent to the pigment. Do not place the skin marker directly on to the pigment. It is only necessary to dot the areas where there is an absence of hair so that there will be no question of where your design is. For example, if there is an absence of hair in the tail area of the eyebrow, and the rest of the brow was perfectly shaped with the hair, then only mark the tail. If the bulb and the arch of the brow need marking then only mark those areas, etc.



3. Whether you mark the entire eyebrow, or just a part of it, it is important to always draw a solid line at the end of the eyebrow to mark its end. This will help to assure the one eyebrow will not be longer than the other.



4. You may now begin the work on the inside of, not directly on, the dots. Make sure that your needle does not touch the dots as this can push the pigment into the skin. As you are working and wiping the pigment away. You will see a dotted shape of an eyebrow.



5. Continue to fill in the eyebrow and by the time you're finished, the dots will be barely visible. You should now have the original shape that you first drew on, except now it's permanent.

